Uro-Tainer® Twin

Suby G B. Braun

BG	Указания за употреба	2	
CZ	Informace pro použití	4	
DE	Gebrauchsanweisung	6	schwarz
DK	Brugsanvisning	8	Dokument = 148 x 210 mm 32 Seiten
ES	Instrucciones de uso	10	Lätus - - -
FI	Käyttöohje	12	
FR	Mode d'emploi	14	Irrigation Solution___814_GIF 814/UF00305/0113
GB	Information for Use	16	GIF (A5-Booklet) Standort Crissier
HU	Használati utasítás	18	
IT	Istruzioni per l'uso	20	
NL	Gebruiksaanwijzing	22	
NO	Informasjon for bruk	24	
PT	Instruções de utilização	26	
SE	Bruksanvisning	28	
SK	Informácie pre používanie	30	

Uro-Tainer® Twin Suby G B. Braun

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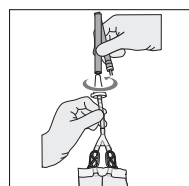
Uro-Tainer® Twin Suby G B. Braun

Maintenance solution for urethral and
suprapubic catheters
Sterile, free of endotoxins

Composition

100 ml solution contains:

Citric monohydrate acid	3.23 g
Light magnesium oxide	0.38 g
Sodium bicarbonate	0.7 g
Disodium edetate 2H ₂ O (excipient) in Water for Injections	0.01 g

Description

A citric acid-based solution for indwelling urethral and suprapubic catheters to dissolve encrustation that could lead to the obstruction of the catheter.

Uro-Tainer® is a ready-to-use, disposable system. It consists of a PVC sachet, in a protective plastic overwrapping, containing 2 chambers of 30 ml of sterile solution, 2 flexible tubes fitted with clamps, and a sterile universal connector protected by a plastic cap with a tamperproof seal.

Dosage

Unless otherwise advised by a doctor, depending on the seriousness of the patient's condition, one application of approximately 10 minutes (5 minutes for each chamber) per day, up to a maximum of twice daily.

Overdosage

No clinical data exists concerning overdosage.

Important information

- This solution is intended for urinary catheter rinsing only : *not for intravenous infusion*.
- Instil the solution by gravity feed, avoid force.
- Precautions should be taken when performing bladder washout on patients with spinal injuries because of the risk of autonomic dysreflexia.
- Use after urinary tract surgery should be evaluated by a physician.
- Not to be used in cases of cystitis or other uro-genital conditions that can produce haematuria (blood in the urine). Refer to a doctor who will decide on the treatment to be followed.
- The solution is not a vehicle for additives. It must not be used for the dissolution of medication.
- The acidity of the Suby G solution (pH ~ 4) can alter the effect of other medication. This should be noted particularly in connection with the treatment of urinary infections with antibiotics.
- Apply using aseptic technique.

- Only to be used if the overwrapping, the contents, and the sealing cap are intact, and the solution is clear.
- This is a single-use system. Do not re-use.
- Dispose of any partially used sachet and any unused solution.
- Discard properly after use.
- Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.
- Animal experiments have shown DEHP to be potentially toxic to reproduction. Given the current state of scientific knowledge, a risk (in case of long-term use) especially for premature male babies cannot be completely excluded. As a precaution, the application of medical products containing DEHP should be restricted to short-term use for pregnant women, breastfeeding mothers, infants, and children.
- Store out of the reach of children.
- Refer to the operating diagram on the next page.

Secondary effects

- Citric acid may cause some patients to experience slight irritation and even temporary pain, a burning sensation, or spasms of the bladder (desire to urinate). In these cases the application should be carried out less frequently, or alternated with an application of NaCl 0.9%. In general, however, this treatment is well tolerated and these secondary effects do not require it to be interrupted.
- If the liquid exits from the urethra during the irrigation, the catheter may no longer be in the bladder and the patient may require a new catheter.
- If the liquid does not flow, the catheter may be kinked in the bladder or it may be blocked and may require changing.

Expiry date

Do not use after the date marked "EXP" on the back of the sachet.

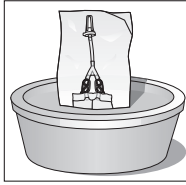
Storage

Store in the original packaging, protected from light, at a temperature not exceeding 25 °C.


Latest information update

06/2010


1. The Uro-Tainer Twin sachet is supplied sterile and should not be removed from its overwrap until ready to use. The solution should be at body temperature, approximately 37 degrees. If the solution requires warming prior to use immerse the wrapped sachet into luke-warm water.

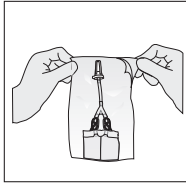
2. Peel open the plastic overwrap and remove the Uro-Tainer Twin.

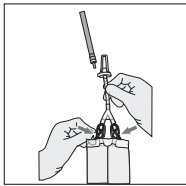
3. Close both tubes with the green and white clamps.

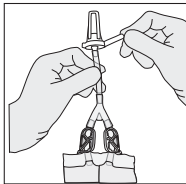
4. Remove the tear away closure ring.

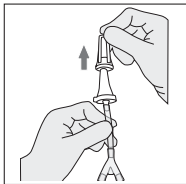
5. Twist and withdraw cap without touching sterile connector. Release the white clamp into the open position and allow a few drops of the solution to flow into the catheter to remove any air locks.

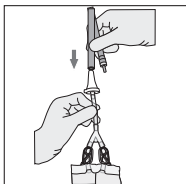
6. Insert connector into the catheter.

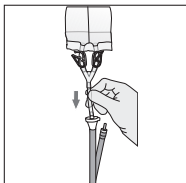
7. Holding Uro-Tainer Twin above the level of the bladder, allow the solution from the 1st chamber to flow into the bladder. To avoid any potential trauma to the bladder do not use force or agitation.

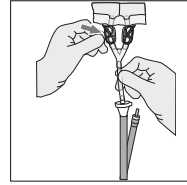
8. Close open tube with white clamp.

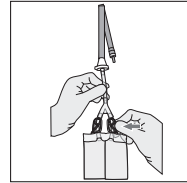
9. After 5 minutes release white clamp and hold bag below the level of the bladder, and drain the solution. Close tube with white clamp.

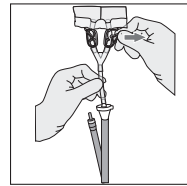
10. Now repeat the process using the 2nd chamber of solution. Hold Uro-Tainer Twin above the level of the bladder and release the green clamp. Allow the solution to flow into the bladder. Again, do not use force or agitation.

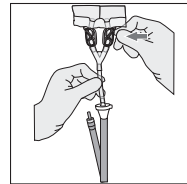
11. Close open tube with green clamp.

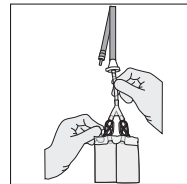12. After 5 minutes release green clamp and hold bag below the level of the bladder, and drain the solution. Close tube with green clamp. Ensure both clamps are now closed.

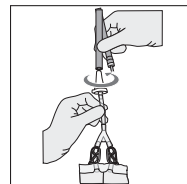13. Remove the connector from the catheter by turning gently. Now connect the catheter to a sterile drainage bag. Discard after treatment. For single use only.

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B. Braun Medical AG