## AQUA B. BRAUN

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## Aqua B. Braun

Irrigation solution, sterile and endotoxin-free

### Composition

1000 ml solution contain:  
Water for Injection 1000 ml

### Indications

For irrigation and cleansing during surgical intervention  
For irrigation of wounds and burns

For moistening wound tamponades, cloths and dressings

For rinsing and cleansing of bladder catheters

For irrigation and cleansing in stoma care

For rinsing and cleansing of instruments and equipment

For filling respiratory air humidifiers

### Dosage

The amount of fluid depends on the clinical condition.

### Overdosage

Overdosage during wound irrigation and intra-operative application can lead to absorption of irrigation fluid, that can lead to hypervolaemia in the form of hypotonic hyperhydration (hyponatraemia).

Headache, nausea, restlessness and disorientation are symptoms of hyperhydration. In severe cases coma-like states can occur.

### Countermeasures in cases of overdosage

Discontinuation of irrigation, correction of the water and electrolyte balance depending

on the particular clinical situation.

### Warnings and precautions

Do not use for infusion.  
Only use the solution if the seal is undamaged and the solution is clear.

Opened containers must not be saved, quantities left over must be discarded.

Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.

Store out of reach of children.  
Aqua B. Braun must not be used for intrathecal irrigation procedures.

### Adverse side effects

None are known.

### Expiry date

The solution must not be used after the expiry date given on the label.

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